

myProbes Passport (ASR Probes)

MPA60251

20cen in Aqua Probe Concentrated
Analyte Specific Reagent


Nucleotide Locations (If requesting more than one probe please specify individual probe colours)

	Chromosome Locus	Start	End	Colour
1	20cen	20p11.1	20q11.1	Aqua

Database Used and Version

GRCh37(hg19)

Probe Map

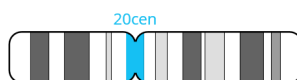

MPA60251 v001

Materials Provided

Probe: 15µl per vial

The probe is provided pre-mixed in hybridisation solution (Formamide; Dextran Sulphate; SSC) and is ready to use. It is directly labelled with a aqua fluorophore, which lies in the Aqua Spectrum. The probe has been batch released after QC testing on Peripheral Blood Sample.

Warnings and Precautions

1. Analyte Specific Reagent. Analytical and performance characteristics are not established.
2. For professional use only.
3. Laboratories must undertake all appropriate validation of any Laboratory Developed Test (LDT) as per the CLIA Regulations.
4. Probe mixtures contain formamide, which is a teratogen; do not breathe fumes or allow skin contact. Handle with care; wear gloves and a lab coat.
5. Follow local disposal regulations for your location along with recommendations in the Safety Data Sheet to determine the safe disposal of this product. This also applies to damaged test kit contents.
6. Dispose of all used reagents and any other contaminated disposable materials following procedures for infectious or potentially infectious waste. It is the responsibility of each laboratory to handle solid and liquid waste according to their nature and degree of hazardousness and to

treat and dispose of them (or have them treated and disposed of) in accordance with any applicable regulations.

7. Operators must be capable of distinguishing the colors red, blue and green.
8. Custom probes are specifically developed for individual customers' Analyte Specific Reagent (ASR) requirements. Therefore, prior to any use of these probes, users should review the design of such probes to confirm they are suitable for their requirements.

Storage and Handling

1. Store the probe between -25°C to -15°C.
2. Based on the stability established for other substantially equivalent CytoCell probes, this myProbes product should be stable for 2 years post manufacture date when stored as indicated on the label.
3. Store the probe vial in the dark. Ensure that exposure of the probe to laboratory lights is limited at all times.

Known Cross-Reactivity

Cross hybridisation may be observed at QC release on the sample type specified to the centromeric regions of chromosomes 2,4,8,9,13,14,15,18,21 and 22.

Customer Support

Please contact the CytoCell Technical Support Department or email probes@cytocell.com.

Patents and Trademarks

myProbes and CytoCell are registered trademarks of Cytocell Ltd.

Cytocell Ltd.

Oxford Gene Technology
418 Cambridge Science Park,
Milton Road,
Cambridge, CB4 0PZ, UK
T: +44(0)1223 294048
E: probes@cytocell.com
W: www.ogt.com

