A Sysmex Group Company

Instructions For Use
REF: LPU 005-S / LPU 005

Prader-Willi/Angelman (SNRPN) Probe

PROFESSIONAL USE ONLY

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Further information available at www.ogt.com

Fluorescence *In Situ* Hybridisation (FISH) is a technique that allows DNA sequences to be detected on metaphase chromosomes or in interphase nuclei from fixed cytogenetic samples. The technique uses DNA probes that hybridise to entire chromosomes or single unique sequences, and serves as a powerful adjunct to classic cytogenetics. Recent developments have meant that this valuable technique can now be applied as an essential diagnostic tool in prenatal, haematological and pathological chromosomal analysis. Target DNA, after fixation and denaturation, is available for annealing to a similarly denatured, fluorescently labelled DNA probe, which has a complementary sequence. Following hybridisation, unbound and non-specifically bound DNA probe is removed and the DNA is counterstained for visualisation. Fluorescence microscopy then allows the visualisation of the hybridised probe on the target material.

Probe Information

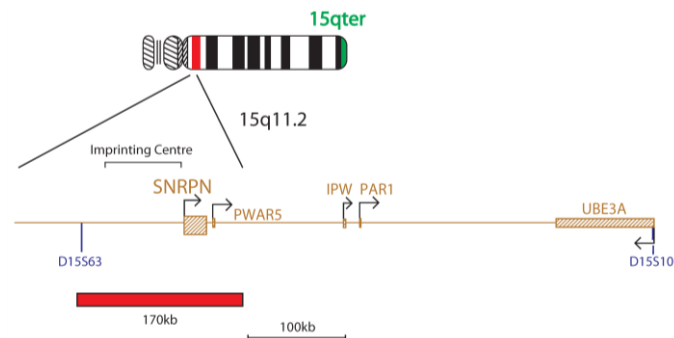
Prader-Willi Syndrome (PWS) and Angelman Syndrome (AS) are distinct neurogenetic disorders caused by the loss of function of genes on chromosome 15 (bands 15q11-13), on either the paternally or maternally inherited chromosome, respectively¹. In 70% of patients, a large interstitial deletion of 3-4Mb is observed^{1,2}. In around 3% of patients, an imprinting defect is observed, caused by either an epimutation or a microdeletion of the Imprinting Centre (IC)^{1,3}. Uniparental disomy, in which both chromosome 15s are inherited from the same parent, accounts for most of the remaining patients with PWS/AS¹.

The SNRPN gene is one of four imprinted loci that are expressed from the paternal chromosome 15 region (15q11-13) and maps to the minimally deleted region (MDR) involved in PWS⁵. Its chromosomal location and imprinting status suggest it plays a possible role in the aetiology of PWS⁴.

The imprinting centre (IC) maps to a 100kb region proximal to SNRPN. Parental deletions or mutations in the IC impair the imprinting process in 15q11-13 and cause one of two distinct diseases in their offspring^{5,6}. Most of the PWS imprinting deletions involve SNRPN and are approximately 200kb in size. The AS imprinting deletions are small (approximately 40kb), involve the BD3 region, and do not include SNRPN.

Probe Specification

SNRPN, 15q11.2, Red
15qter, 15q26.3, Green

The Prader-Willi/Angelman (SNRPN) probe is 170kb, labelled in red and covers the whole SNRPN gene as well as entire imprinting centre. The 15qter subtelomere specific probe (clone 154P1), labelled in green, allows identification of chromosome 15 and acts as a control probe.

Materials Provided

Probe: 50µl per vial (5 tests) or 100µl per vial (10 tests)
Amount of red SNRPN probe: 22-27ng/test
Amount of green 15qter probe: 27-34ng/test
The probes are provided premixed in hybridisation solution (Formamide; Dextran Sulphate; SSC) and are ready to use.

Counterstain:

150µl per vial (15 tests)
The counterstain is DAPI antifade (ES: 0.125µg/ml DAPI (4,6-diamidino-2-phenylindole)).

Warnings and Precautions

1. For *in vitro* diagnostic use. For professional use only.
2. Wear gloves when handling DNA probes and DAPI counterstain.
3. Probe mixtures contain formamide, which is a teratogen; do not breathe fumes or allow skin contact. Wear gloves, a lab coat, and handle in a fume hood. Upon disposal, flush with a large volume of water.
4. DAPI is a potential carcinogen. Handle with care; wear gloves and a lab coat. Upon disposal, flush with a large volume of water.
5. All hazardous materials should be disposed of according to your institution's guidelines for hazardous waste disposal.

Storage and Handling

The kit should be stored between -25°C to -15 °C in a freezer until the expiry date indicated on the kit label. The probe and counterstain vials must be stored in the dark.

Equipment Necessary but not Supplied

1. Hotplate (with a solid plate and accurate temperature control up to 80°C).
2. Variable volume micropipettes and tips range 1µl - 200µl.
3. Water bath with accurate temperature control at 72°C.
4. Microcentrifuge tubes (0.5ml).
5. Fluorescence microscope (Please see Fluorescence Microscope Recommendation section).
6. Plastic or glass coplin jars.
7. Forceps.
8. Fluorescence grade microscope lens immersion oil.
9. Bench top centrifuge.
10. Microscope slides.
11. 24x24mm coverslips.
12. Timer.
13. 37°C incubator.
14. Rubber solution glue.

Fluorescence Microscope Recommendation

For optimal visualisation of the probe we recommend a 100-watt mercury lamp and plan apochromat objectives x63 or x100. The Triple bandpass filter DAPI/FITC/Texas Red is optimal for viewing all fluorophores and DAPI simultaneously.

Sample Preparation

The kit is designed for use on cultured peripheral blood cells fixed in Carnoy's fixative, that should be prepared according to the laboratory or institution guidelines. Prepare air dried samples on microscope slides according to standard cytogenetic procedures.

FISH Protocol

(Note: Please ensure that exposure of the probe to laboratory lights is limited at all times).

Slide preparation

1. Spot the cell sample onto a glass microscope slide. Allow to dry.
2. Immerse the slide in 2xSSC for 2 minutes at room temperature (RT) without agitation.
3. Dehydrate in an ethanol series (70%, 85% and 100%), each for 2 minutes at RT.
4. Allow to dry.

Pre-Denaturation

5. Remove the probe from the freezer and allow it to warm to RT.
6. Ensure that the probe solution is uniformly mixed with a pipette.
7. Remove 10µl of probe per test, and transfer it to a microcentrifuge tube. Quickly return the remaining probe to the freezer.
8. Place the probe and the sample slide to prewarm on a 37°C (+/- 1°C) hotplate for 5 minutes.
9. Spot 10µl of probe mixture onto the cell sample and carefully apply a coverslip. Seal with rubber solution glue and allow the glue to dry completely.

Denaturation

10. Denature the sample and probe simultaneously by heating the slide on a hotplate at 75°C (+/- 1°C) for 2 minutes.

Hybridisation

11. Place the slide in a humid, lightproof container at 37°C (+/- 1°C) overnight.

Post-Hybridisation Washes

12. Remove the coverslip and all traces of glue carefully.
13. Immerse the slide in 0.4xSSC (pH 7.0) at 72°C (+/- 1°C) for 2 minutes without agitation.
14. Drain the slide and immerse it in 2xSSC, 0.05% Tween-20 at RT (pH 7.0) for 30 seconds without agitation.
15. Drain the slide and apply 10µl of DAPI antifade onto each sample.
16. Cover with a coverslip, remove any bubbles and allow the colour to develop in the dark for 10 minutes.
17. View with a fluorescence microscope.

Stability of Finished Slides

FISHed slides remain analysable for up to 1 month if stored in the dark at/or below RT.

Procedural Recommendations

1. Baking or ageing of slides is not recommended as it may reduce signal fluorescence.
2. Hybridisation conditions may be adversely affected by the use of reagents other than those provided or recommended by CytoCell Ltd.
3. The use of a calibrated thermometer is strongly recommended for measuring temperatures of solutions, waterbaths, and incubators as these temperatures are critical for optimum product performance.
4. The wash concentrations, pH and temperatures are important as low stringency can result in non-specific binding of the probe and too high stringency can result in a lack of signal.
5. Incomplete denaturation can result in lack of signal and over denaturation can also result in non-specific binding.

Expected Results

In a normal cell, there should be two red and two green signals (2R, 2G). A cell with a deletion of SNRPN should have one red signal and two green controls (1R, 2G).

Limitations

Reporting and interpretation of FISH results should be consistent with professional standards of practice and should take into consideration other clinical and diagnostic information. This kit is intended as an adjunct to other diagnostic laboratory tests and therapeutic action should not be initiated on the basis of the FISH result alone. This device is designed to detect genomic losses larger than the region covered by the red clone in this probe set. Genomic losses outside this region or partial losses of this region may not be detected with this product.

Additional Information

For additional product information please contact the CytoCell Technical Support Department.

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Patents and Trademarks
CytoCell is a registered trademark of CytoCell Ltd.