A Sysmex Group Company

Instructions For Use  
REF: LPU 019-S/LPU 019

## Smith-Magenis (RAI1)/Miller-Dieker Probe Combination

PROFESSIONAL USE ONLY

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Further information available at [www.ogt.com](http://www.ogt.com)

Fluorescence *In Situ* Hybridisation (FISH) is a technique that allows DNA sequences to be detected on metaphase chromosomes or in interphase nuclei from fixed cytogenetic samples. The technique uses DNA probes that hybridise to entire chromosomes or single unique sequences, and serves as a powerful adjunct to classic cytogenetics. Recent developments have meant that this valuable technique can now be applied as an essential diagnostic tool in prenatal, haematological and pathological chromosomal analysis. Target DNA, after fixation and denaturation, is available for annealing to a similarly denatured, fluorescently labelled DNA probe, which has a complementary sequence. Following hybridisation, unbound and non-specifically bound DNA probe is removed and the DNA is counterstained for visualisation. Fluorescence microscopy then allows the visualisation of the hybridised probe on the target material.

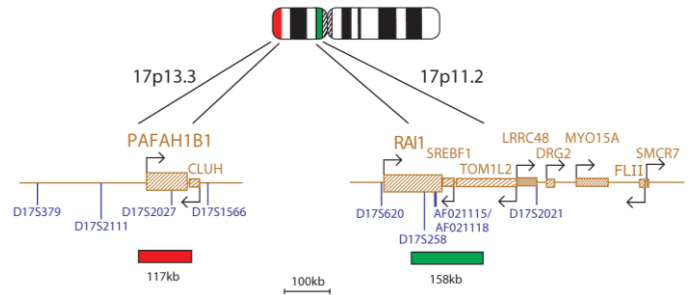
### Probe Information

Smith-Magenis syndrome (SMS) is a multiple congenital anomaly syndrome characterised by mental retardation, neurobehavioral abnormalities, sleep disturbances, short stature, minor craniofacial and skeletal anomalies, congenital heart defects and renal anomalies<sup>1,2</sup>. It is one of the most frequently observed human microdeletion syndromes and is associated with an interstitial deletion of the chromosome band 17p11.2<sup>2</sup>. Molecular studies in SMS patients suggest a minimally deleted region (MDR) spanning approximately 700kb<sup>3,5</sup>, though the common deletion is around 4Mb in size<sup>4</sup>. The proximal boundary of the MDR is within a region of overlap between the FLII and LLGL1 genes, and the distal boundary within the PEMT gene<sup>3</sup>. Deletions or mutations in RAI1 (Retinoic Acid Induced 1) gene, which lies within the MDR, are associated with the syndrome<sup>3,5,6,7</sup>. RAI1 was shown to be the primary gene responsible for most features of SMS<sup>8,9</sup>. Whilst deletion of the 17p11.2 region results in SMS, duplication of the same region results in a similar, yet distinct, disorder known as Potocki-Lupski syndrome<sup>10</sup>. Phenotypically this shares many similarities to SMS, though it is generally milder, but does have some unique clinical findings<sup>10</sup>. The common duplication involves the same 4Mb region as the SMS deletion as both syndromes are mediated by non-allelic homologous recombination between flanking low copy repeat regions<sup>4</sup>.

Miller-Dieker syndrome (MDS) is a multiple malformation characterised by classical lissencephaly, a characteristic facial appearance and sometimes other birth defects<sup>11</sup>. It is associated with visible or submicroscopic rearrangements within chromosome band 17p13.3 in almost all cases<sup>12</sup>. Isolated lissencephaly sequence (ILS) consists of classical lissencephaly with no other major anomalies<sup>13</sup>. Submicroscopic deletions of chromosome 17p13.3 have been detected in almost 40% of these patients<sup>12</sup>. MDS is considered a contiguous gene deletion syndrome where deletion of physically contiguous genes leads to the complex phenotypic abnormalities observed. The PAFAH1B1 (LIS1) gene is located at 17p13.3 and is recognised as the causative gene for the lissencephaly phenotype in both MDS and ILS<sup>14,15</sup>. Deletions in MDS patients always include the PAFAH1B1 gene, together with other telomeric loci to a distance in excess of 250kb<sup>14</sup>.

### Probe Specification

LIS1 (PAFAH1B1), 17p13.3, Red  
RAI1, 17p11.2, Green

The Miller-Dieker (LIS1) probe is 117kb, labelled in red and covers the entire LIS1 (PAFAH1B1) gene. The Smith-Magenis (RAI1) probe is 158kb, labelled in green and covers the centromeric end of the RAI1 gene and includes the D17S258 marker. The two unique sequences act as control probes for each other and allow identification of chromosome 17.

### Materials Provided

**Probe:** 50µl per vial (5 tests) or 100µl per vial (10 tests)

Amount of red LIS1 (PAFAH1B1) probe: 23-29ng/test

Amount of green RAI1 probe: 9-11ng/test

The probes are provided premixed in hybridisation solution (Formamide; Dextran Sulphate; SSC) and are ready to use.

**Counterstain:** 150µl per vial (15 tests)

The counterstain is DAPI antifade (ES: 0.125µg/ml DAPI (4,6-diamidino-2-phenylindole)).

### Warnings and Precautions

1. For *in vitro* diagnostic use. For professional use only.
2. Wear gloves when handling DNA probes and DAPI counterstain.
3. Probe mixtures contain formamide, which is a teratogen; do not breathe fumes or allow skin contact. Wear gloves, a lab coat, and handle in a fume hood. Upon disposal, flush with a large volume of water.
4. DAPI is a potential carcinogen. Handle with care; wear gloves and a lab coat. Upon disposal, flush with a large volume of water.
5. All hazardous materials should be disposed of according to your institution's guidelines for hazardous waste disposal.

### Storage and Handling

The kit should be stored between -25°C to -15°C in a freezer until the expiry date indicated on the kit label. The probe and counterstain vials must be stored in the dark.

### Equipment Necessary but not Supplied

1. Hotplate (with a solid plate and accurate temperature control up to 80°C).
2. Variable volume micropipettes and tips range 1µl - 200µl.
3. Water bath with accurate temperature control at 72°C.
4. Microcentrifuge tubes (0.5ml).
5. Fluorescence microscope (Please see Fluorescence Microscope Recommendation section).
6. Plastic or glass coplin jars.
7. Forceps.
8. Fluorescence grade microscope lens immersion oil.
9. Bench top centrifuge.
10. Microscope slides.
11. 24x24mm coverslips.
12. Timer.
13. 37°C incubator.
14. Rubber solution glue.

### Fluorescence Microscope Recommendation

For optimal visualisation of the probe we recommend a 100-watt mercury lamp and plan apochromat objectives x63 or x100. The Triple bandpass filter DAPI/FITC/Texas Red is optimal for viewing all fluorophores and DAPI simultaneously.

### Sample Preparation

The kit is designed for use on cultured peripheral blood cells fixed in Carnoy's fixative that should be prepared according to the laboratory or institution guidelines. Prepare air dried samples on microscope slides according to standard cytogenetic procedures.

### FISH Protocol

(Note: Please ensure that exposure of the probe to laboratory lights is limited at all times).

### Slide preparation

1. Spot the cell sample onto a glass microscope slide. Allow to dry.
2. Immerse the slide in 2xSSC for 2 minutes at room temperature (RT) without agitation.
3. Dehydrate in an ethanol series (70%, 85% and 100%), each for 2 minutes at RT.
4. Allow to dry.

### Pre-Denaturation

5. Remove the probe from the freezer and allow it to warm to RT.
6. Ensure that the probe solution is uniformly mixed with a pipette.

- Remove 10µl of probe per test, and transfer it to a microcentrifuge tube. Quickly return the remaining probe to the freezer.
- Place the probe and the sample slide to prewarm on a 37°C (+/- 1°C) hotplate for 5 minutes.
- Spot 10µl of probe mixture onto the cell sample and carefully apply a coverslip. Seal with rubber solution glue and allow the glue to dry completely.

#### Denaturation

- Denature the sample and probe simultaneously by heating the slide on a hotplate at 75°C (+/- 1°C) for 2 minutes.

#### Hybridisation

- Place the slide in a humid, lightproof container at 37°C (+/- 1°C) overnight.

#### Post-Hybridisation Washes

- Remove the coverslip and all traces of glue carefully.
- Immerse the slide in 0.4xSSC (pH 7.0) at 72°C (+/- 1°C) for 2 minutes without agitation.
- Drain the slide and immerse it in 2xSSC, 0.05% Tween-20 at RT (pH 7.0) for 30 seconds without agitation.
- Drain the slide and apply 10µl of DAPI antifade onto each sample.
- Cover with a coverslip, remove any bubbles and allow the colour to develop in the dark for 10 minutes.
- View with a fluorescence microscope.

#### Stability of Finished Slides

FISHed slides remain analysable for up to 1 month if stored in the dark at/or below RT.

#### Procedural Recommendations

- Baking or ageing of slides is not recommended as it may reduce signal fluorescence.
- Hybridisation conditions may be adversely affected by the use of reagents other than those provided or recommended by CytoCell Ltd.
- The use of a calibrated thermometer is strongly recommended for measuring temperatures of solutions, waterbaths, and incubators as these temperatures are critical for optimum product performance.
- The wash concentrations, pH and temperatures are important as low stringency can result in non-specific binding of the probe and too high stringency can result in a lack of signal.
- Incomplete denaturation can result in lack of signal and over denaturation can also result in non-specific binding.

#### Expected Results

In a normal cell, there should be two red and two green signals (2R, 2G). A cell with a deletion of the LIS1 (PAFAH1B1) probe target should have one red signal and two green controls (1R, 2G) and a cell with a deletion of the RAI1 probe target should have two red controls and one green signal (2R, 1G).

#### Limitations

Reporting and interpretation of FISH results should be consistent with professional standards of practice and should take into consideration other clinical and diagnostic information. This kit is intended as an adjunct to other diagnostic laboratory tests and therapeutic action should not be initiated on the basis of the FISH result alone.

#### Additional Information

For additional product information please contact the CytoCell Technical Support Department.

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#### Patents and Trademarks
CytoCell is a registered trademark of CytoCell Ltd.