

myProbes Passport (ASR Probes)

MPH18982
17cen in FITC Spectrum
Analyte Specific Reagent


Nucleotide Locations (If requesting more than one probe please specify individual probe colours)

	Chromosome Locus	Start	End	Colour
1	17cen	17p11.1	17q11.1	Green

Database Used and Version

GRCh37 (hg19)

Probe Map

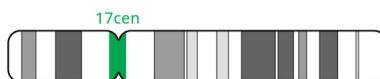

MPH18982 v001

Materials Provided

Probe: 250µl per vial

The probe is provided pre-mixed in hybridisation solution (Formamide; Dextran Sulphate; SSC) and is ready to use. It is directly labelled with a green fluorophore, which lies in the FITC Spectrum. The probe has been batch released after QC testing on Bone Marrow Sample.

Warnings and Precautions

1. Analyte Specific Reagent. Analytical and performance characteristics are not established.
2. For professional use only.
3. Laboratories must undertake all appropriate validation of any Laboratory Developed Test (LDT) as per the CLIA Regulations.
4. Probe mixtures contain formamide, which is a teratogen; do not breathe fumes or allow skin contact. Handle with care; wear gloves and a lab coat.
5. Follow local disposal regulations for your location along with recommendations in the Safety Data Sheet to determine the safe disposal of this product. This also applies to damaged test kit contents.
6. Dispose of all used reagents and any other contaminated disposable materials following procedures for infectious or potentially infectious waste. It is the responsibility of each laboratory to handle solid and liquid waste according to their nature and degree of hazardousness and to treat and dispose of them (or have them treated and disposed of) in accordance with any applicable regulations.
7. Operators must be capable of distinguishing the colors red, blue and green.

8. Custom probes are specifically developed for individual customers' Analyte Specific Reagent (ASR) requirements. Therefore, prior to any use of these probes, users should review the design of such probes to confirm they are suitable for their requirements.

Storage and Handling

1. Store the probe between -25°C to -15°C.
2. Based on the stability established for other substantially equivalent CytoCell probes, this myProbes product should be stable for 2 years post manufacture date when stored as indicated on the label.
3. Store the probe vial in the dark. Ensure that exposure of the probe to laboratory lights is limited at all times.

Known Cross-Reactivity

Cross hybridisation may be seen to the centromeric region of Chromosome 11.

Customer Support

Please contact the CytoCell Technical Support Department or email probes@cytoCELL.com.

Patents and Trademarks

myProbes and CytoCell are registered trademarks of CytoCell Ltd.

This product contains technology licensed from Life Technologies Corporation that is available for human diagnostics or life science research use only.

CytoCell Ltd.

Oxford Gene Technology
418 Cambridge Science Park,
Milton Road,
Cambridge, CB4 0PZ, UK
T: +44(0)1223 294048
F: +44(0)1223 294986
E: probes@cytoCELL.com
W: www.ogt.com

